
ENG
Maxxi Gold Electrodes
User Instructions

Intentions of Use: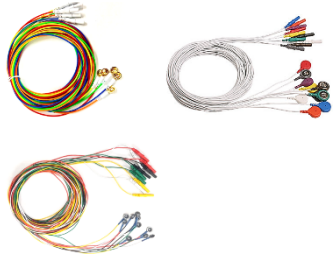

The Maxxi Gold Electrode is intended for non-invasive use as recording electrodes in studies of physiological signals. This product is indicated for use with adult or pediatric patients during electroencephalography, including biofeedback, sleep studies and evoked potential recordings.

Maxxi Gold electrodes Sensor Instructions for Use:

1. First, gently abrade the skin with a FDA pre-approved abrasive product such as Nuprep.
2. Use Maxxi Gold Electrode applying just enough FDA pre-approved conductive paste inside of the electrode to very slightly overfill the cup.
3. Place the electrode onto the electrode site and press with medium pressure. A small amount of paste may come out of the hole. Pressing too hard will cause all of the paste to come out and the electrode will not adhere well. Use either tape or a postage stamp size piece of gauze to fix the electrode in place.
4. If the electrode site has hair around it, use a cut up piece of gauze the size of a postage stamp and press onto the electrode. The paste that came out of the hole when it was pressed into place will hold the gauze square. If no paste comes out of the hole, place a small amount of Conductive paste on the gauze and press the gauze onto the electrode using the paste to make it stick.
5. If the electrode site does not have hair around it, use adhesive tape such as 1" Micropore tape to secure the electrodes. Use the same technique as above pressing the electrode into place. Use 3 or 4 cm of tape (1 1/4") to cover the electrode.
6. Check the impedance and start the study.

Maxxi Gold Snap electrodes Sensor Instructions for Use:

1. First, gently abrade the skin with an FDA pre-approved abrasive product such as Nuprep.
2. Use Maxxi Gold Snap Electrode applying FDA pre-approved adhesive electrode.
3. Place the adhesive electrode on the patient site and press with medium pressure.
4. Connect the Snap button electrode to the adhesive electrode.
5. Check the impedance and start the study.

Recommended polysomnography filter settings:

6. Low-cut filter: 0.1 Hz.
7. High-cut filter: 15 Hz.

Cleaning (Not applicable for disposable):

1. Wipe the wires with a non-corrosive (to plastic) cleanser to clean before use.
2. Make sure the wires are thoroughly dry before reusing it.
3. Do not soak the parts in disinfectants
4. Do not stretch the cable.
5. Do not wet the connectors.
6. Do not leave the electrode immersed in water for more than 5 minutes.

Warnings and precautions:

1. Do not connect it directly to the electric grid (RISK OF ELECTRIC SHOCK).
2. THIS DEVICE IS NOT TO BE USED IN MR ENVIRONMENT.
3. Caution: Federal law restricts this device to sale by or on the order of a physician.
4. Medical product NON-STERILE. - Do not use the product if the package is previously opened or violated.
5. Maxxi GOLD Electrode is for medical professional use only such, physicians and trained medical technologists.
6. Maxxi GOLD Electrode may be used only in conjunction with an approved recording system.
7. Discontinue use if sensor shows signs of wear, damage, or exposed metal.
8.  For correct recycling process, please follow local regulations to dispose of the electrical material.
9. Reusable electrodes, please discontinue use after 12 months from the first usage date.
10.  Single use electrode, it is not intended to use multiple times.

Operating/Storage Conditions:

Operating conditions	5°C (40°F) - 40°C (104°F)
Storage temperature	-20°C (-4°F) - 60°C (140°F)
Operating and storage humidity	5% - 95% (Non-condensing)

If any physical disturbance and/or chemical breakdown happens during the storage period, do not use the product.

Warranty Period:

Neurovirtual USA warrants the electrodes to be free of defects in materials and workmanship for a period of 6 months from the date purchased.

Neurovirtual USA warrants the disposable electrodes to be free of defects in materials and workmanship for a period of 3 months from the date purchased.

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Neurovirtual USA Inc.